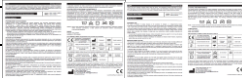
KARTA PRODUKTU

PKT862B

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DISCEKNN.RV.0/12.2023 | PL

INSTRUCTIONS FOR USE

The device should be used to cover the whole surface or part of the body, during rest or sleep, for allergy sufferers and healthy people. To ensure proper hygiene conditions, always use the device covered with a cotton case.

The device is a non-invasive medical device intended for repeated use, with limited exposure (maximum daily usage of 10 to 12 hours). In case of prolonged use, it is necessary to aerate the device. The product must be used in accordance with its intended use and other information provided in these Instructions and on the product label.

CONTRA-INDICATIONS FOR USE

Do not use in case of people who are allergic to polyester. Do not use the product in direct contact with the skin. Do not use in children under 3 years of age. Do not use in case of skin disorders, other allergic symptoms and in persons with damaged skin, open wounds not protected by a dressing and/or ulcerations.

WARNINGS

The device is an item of multiple, personal use. For health and hygiene reasons, the device must not be used by more than one person. The device is not intended for use by children. It is mandatory to carry out cleaning of the device and the case before use.

Always use the device covered by the device case.

Do not store or use a damaged or a wet device. Do not use the device in damp or dusty rooms.

Do not use the device if it is damaged, misshapen or heavily (dirty) soiled. In the event of damage or discolouring, the device must be removed, the device must be replaced by a new one.

Due to the effect of wear and tear or the washing process, it is possible for the cover to become pilled or heavily soiled. The device must be replaced by a new one.

Do not use the device for a longer period of use than indicated on the device.

Due to the quality period of use of the device, the device may over time lose its original properties, provided that the device is used in accordance with the instructions and product labels.

Do not use the device for a longer period of use than indicated on the device. The packaging should be removed so that it is not within the reach of children.

In the event of allergic reactions or any other adverse reactions associated with use of the device, discontinue use immediately and consult a doctor.

Cover: 100% polyester.
Filling: 100% polyester.

EXPLANATION OF SYMBOLS INDICATING THE DEVICE MAINTENANCE PROCESS					
	Maximum washing temperature of 60°C, normal process		Do not bleach		Do not dry clean.
	Tumble drying possible, drying at low temperature, exhaust temperature maximum 60°C		Do not iron		

in case of any serious incident related to the device, the manufacturer should be immediately informed in writing to the following address: "John Cotton Europe" Sp. z o.o., ul. John Cotton Europe 1, 59-310 Szczawno-Zdrój, Poland, by phone: +48 74 84 56 448 or electronically at the e-mail address: orders@johncotton.eu. It is also necessary to notify the competent authority of the Member State on whose territory the incident occurred.

 "John Cotton Europe" Sp. z o.o.
ul. John Cotton Europe 1
59-310 Szczawno-Zdrój, Poland
tel.: +48 74 84 56 446 | orders@johncotton.eu
www.johncotton.eu

IJUCEKON/Rev.D12.2023 | EN

IUUCE/KON/Rev.D/12.2023 | EN