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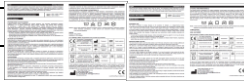
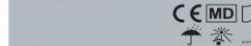
DISCEKNN.RV.0/12.2023 | PL

GB | DUVET WITH COLOR PRINT
160 x 200 cm • Medical device • Made of soft-touch microfibre • Soft, and fluffy – contributes to good comfort while sleeping • Minimises the development of house dust mites and their allergens – suitable for all sufferers • Made with PerfectFit technology – perfectly fits the duvet cover

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5 905428 0369

DUVET WITH PRINT REF KKK 1 | KKK 2 | KKK 3
KKK 4 | KKK 5

Medical device

INTENDED USE

The basis of treatment of allergic diseases is the avoidance of allergens, i.e., allergy-inducing factors. It has been proven that domestic dust mite, which is the main human allergen, do not proliferate on polyester surface - thus a duvet with such content provides a barrier material against this allergen. The Beigen duvet is indicated to be used to support¹ prevention and alleviation of the course of the well mentioned allergic diseases²; as one of the elements of multi treated measures, intended to reduce the exposure to allergens: allergic rhinitis, bronchial asthma, atopic dermatitis, other diseases of allergic type (e.g., conjunctivitis, sinusitis, laryngitis, pharyngitis and palatine tonsillitis).

^{***} According to the International Statistical Classification of Diseases and Related Health Problems; respectively, ICD-10: J30, J45, L20, H10, J32, J37, J31, J35.

INSTRUCTIONS FOR USE
The duvet should be used to cover the whole surface or part of the body, during rest or sleep, for allergy sufferers and healthy people. To ensure proper hygiene conditions, always use the device covered with a cotton case.

The duvet is a non-invasive medical device intended for repeated use, with limited exposure (maximum daily usage of 10 to 12 hours). In case of prolonged use, it is necessary to aerate the device. The product must be used in accordance with its intended use and other information provided in these instructions and on the product labels.

CONTRA-INDICATIONS FOR USE
Do not use in case of people who are allergic to polyester. Do not use the product in direct contact with the skin.
Do not use in children under 3 years of age. Do not use in case of skin conditions, other allergic symptoms and in persons with damaged skin, open wounds not protected by a dressing and/or ulcerations.

The device is an item of multiple, personal use. For health and hygiene reasons, the device must not be used simultaneously or subsequently by others. In an event of a need for other people to use the device, it is mandatory to carry out laundering of the device and the case before use.

Always use the device covered with an additional case.

Do not sleep or use a alarm and/or a seat device. Do not use the device in flames or near flames.

Do not use the device if it is damaged, mismatched or dirty (heavily soiled). In the event of damage or heavy soiling that cannot be removed, the device should be replaced.

Due to the effect of wear and tear or the washing process, it is possible for the cover to become pilled or discoloured (yellowing), and the device fitting may become loose.

The guaranteed period of use of the device is 2 years without losing its original properties, provided that the device is used in accordance with the instructions and product labels.

Plastic packaging can be dangerous for children. After unwrapping the device, the packaging should be removed so that it is not within the reach of children.

In the event of any side effects or any other adverse reactions associated with use of the device, discontinue use immediately and consult a doctor.

of low temperatures and moisture. Protect the device from excessive exposure to dust, direct sunlight and heat sources (e.g. radiators, heaters, heating stoves). The device should be subjected to at least 2-3 maintenance processes per year. Do not store the device in a soiled or damp condition. Do not laundry with other types of products.

After the washing process, the duvet must be firmly shaken. The following maintenance process parameters should be used:

NOTE: In order to eliminate allergens, the device should be washed periodically at high temperature and periodically subjected to low temperatures (freezing); the use of acaricides is also recommended.

COMPOSITION
Cover: 100% polyester.
Filling: 100% polyester.

	CE marking (confirmation of compliance with Regulation (EU) 2017/745 of the European		Manufacturer		Medical device
---	--	---	--------------	---	----------------

	Consult instructions for use		Catalogue number		Batch code
	Keep away from sunlight		Store in dry place		Temperature limit

EXPLANATION OF SYMBOLS INDICATING THE DEVICE MAINTENANCE PROCESS					
	Maximum washing temperature of 60°C, normal process		Do not bleach		Do not dry clean.
	Tumble drying possible, drying				

	at low temperature, exhaust temperature maximum 65°C		Do not iron	
---	--	---	-------------	--

in case of any serious incident related to the device, the manufacturer should be immediately informed in writing to the following address: "John Cotton Europe" Sp. z o.o., ul. John Cotton Europe 1, 56-310 Szczesawo-Zdrój, Poland.

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