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English **Instructions for use**

Please read this instruction carefully before using the device. The instructions provide informations on correct use of the device. The device should be used in accordance with all the informations contained in this instruction. If in doubt regarding the device's use and in any case of suspected allergic reaction, please consult your doctor.

DUVET WITH PRINT	REF KKN 1 KKN 2 KKN 3 KKN 4 KKN 5
Medical device	

INTENDED USE

The basis of treatment of allergic diseases is the avoidance of allergens, i.e., allergy-inducing factors. It has been proven that domestic dust mite, which is the main human allergen, do not proliferate on polyester surfaces - thus a duvet with such content provides a barrier material against this allergen. The Bergeen duvet is intended to be used to support prevention and alleviation of the course of the below mentioned allergic diseases¹, as one of the elements of multi focused measures, intended to reduce the exposure to allergens: allergic rhinitis, bronchial asthma, atopic dermatitis, other diseases of allergic type (e.g., conjunctivitis, sinusitis, laryngitis, pharyngitis and

^{***} According to the International Statistical Classification of Diseases and Related Health Problems; respectively ICD-10: J30, J45, L20, H10, J32, J37, J31, J35.

INSTRUCTIONS FOR USE

The duvet should be used to cover the whole surface or part of the body, during rest or sleep, for allergy sufferers and healthy people. To ensure proper hygiene conditions, always use the device covered with a cotton case. The duvet is a non-invasive medical device intended for repeated use, with limited exposure (maximum daily use of 10 to 12 hours). In case of prolonged use, it is necessary to aerate the device. The product must be used in accordance with its intended use and other information provided in these instructions and on the product label.

CONTRA-INDICATIONS FOR USE
Do not use in case of people who are allergic to polyester. Do not use the product in direct contact with the skin.
Do not use in children under 3 years of age. Do not use in case of skin conditions, other allergic symptoms and in persons with damaged skin, open wounds not protected by a dressing and/or ulcerations.

WARNINGS
The device is an item of multiple, personal use. For health and hygiene reasons, the device must not be used simultaneously or subsequently by others. In an event of a need for other people to use the device, it is mandatory to carry out laundering of the device and the case before use.

Always use the device covered with an additional case.

Do not store or use a damp and/or wet device. Do not use the device in damp or dusty interiors.

Do not use the device if it is damaged, misshaped or dirty (heavily soiled). In the event of damage or heavy soiling that cannot be removed, the device should be replaced.

Due to the effect of wear and tear of the washing process, it is possible for the cover to become pilled and discoloured (yellowing), and the device filling may undergo lumping.

The guaranteed period of use of the device is 2 years without using its original properties, provided that the device is used in accordance with the instructions and product labels.

Mastic packaging can be dangerous for children. After unwrapping the device, the packaging should be removed.

50,000 hours of use - the most of children

In the event of allergic reactions or any other adverse reactions associated with use of the device, discontinue use immediately and consult a doctor.

STORAGE AND MAINTENANCE

The product should be transported in its original packaging. During transport and storage, the product should be protected against dust, dust, moisture and mechanical damage. Store the product at temperatures between 15°C and 30°C, in a dry, well-ventilated place, free of moisture and excessive dust. Do not expose the device to the effects of low temperatures and moisture. Protect the device from excessive exposure to dust, direct sunlight and heat sources (e.g. radiators, heaters, heating stoves). The device should be subjected to at least 2-3 maintenance processes per year. Do not store the device in a soiled or damp condition. Do not laundry with other types of products. After the washing process, the duvet must be firmly shaken. The following maintenance process parameters should be used:

NOTE: In order to eliminate allergens, the device should be washed periodically at high temperature and periodically subjected to low temperatures (freezing); the use of acaricides is also recommended.

COMPOSITION
Cover: 100% polyester.
Filling: 100% polyester.

EXPLANATION OF SYMBOLS USED ON DEVICE LABELS

	CE marking (confirmation of compliance with Regulation (EU) 2017/45 of the European Parliament and of the Council)		Manufacturer		Medical device
	Consult instructions for use		Catalogue number		Batch code
	Keep away from sunlight		Store in dry place		Temperature limit

EXPLANATION OF SYMBOLS INDICATING THE DEVICE MAINTENANCE PROCESS

	Maximum washing temperature of 60°C, normal process		Do not bleach		Do not dry clean.
	Tumble drying possible, drying at low temperature, exhaust temperature maximum 60°C		Do not iron		

In case of any serious incident related to the device, the manufacturer should be immediately informed in writing, to the following address: "John Cotton Europe" Sp. z o.o., ul. John Cotton Europe 1, 58-310 Szczawno-Zdrój, Poland by phone: +48 74 84 56 46 or electronically at the e-mail address: orders@johncotton.eu; It is also necessary to notify the competent authority of the Member State on whose territory the incident occurred.

 "John Cotton Europe" Sp. z o.o.
ul. John Cotton Europe 1
58-310 Szczawno-Zdrój, Poland
tel.: +48 74 84 56 446 | orders@johncotton.eu
www.johncotton.eu

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