KARTA PRODUKTU

K867B

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English

Please read this instruction carefully before using the device. The instructions provide information on correct use of the device and its use should be used in accordance with all of the information contained in the instructions. If you intend changing the device use and in any case of suspected device failure, please consult your doctor.

Instructions for use

CLOUD OF LUXURY DUVEET

REF KCL 1

Medical device

INTENDED USE

The basic treatment of allergic diseases is the avoidance of allergens (i.e., allergen-producing substances). It has been proven that domestic dust mites, which is the main human allergen, does not penetrate on polyester surface. The duvet with duvet cover provides a barrier against allergen against the duvet. The Cloud of Luxury Duvet is indicated to be a "preventive" preparation and treatment of the cause of the bed-mite-mediated allergic diseases, as well as the removal of mite-related measures, intended to reduce exposure to allergens such as dust mites, linens, animal dander, animal dander, other allergens of allergic type (e.g., pollen, fungi, moulds, allergens and volatile substances).

In many cases of suspected allergic diseases, medical consultation should be sought for the device after approval or recommendation of specialists of allergic diseases.

Indicated for the treatment of allergic diseases and related health problems: respectively, ICID-10: J28.0-L28.1, H10, J32, J37, J39, J45.

INSTRUCTIONS FOR USE

The duvet should be used to cover the whole surface of the bed (the body, during or after sleep), for allergic sufferers and healthy people. To remove pollen grains, allergens or the device should be washed with a suitable water.

The duvet is a non-medical medical device intended for repeated use, with limited exposure to the product (maximum of 10 to 15 years of use). The duvet is a non-medical device, it is recommended to use the duvet in accordance with its intended use and other information provided with this instruction on the product label.

CONTRA INDICATIONS FOR USE

The duvet is not a cause of allergic diseases, it is not an allergen. Do not use the product in direct contact with the skin. Do not use in children under 3 years of age. Do not use in cases of skin conditions, allergic diseases, allergic symptoms and persons with damaged skin, open wounds or lesions caused by a healing or another irritation.

WARNINGS

The device is an artificial, personal use. For healthy and healthy people, the device must not be used unnecessarily or abusively by others. In an event of a need for other people to use the device, it is mandatory to carry out thorough cleaning and the device before use.

Do not use the device in the presence of children or pets.

Do not store on or a damp and/or a wet bed. Do not use the device in damp and/or dirty interiors.

Do not use the device if it is damaged, repaired or slightly deformed. In the event of a change or heavy usage, it is recommended to replace the device should be replaced.

Due to the effect of wear and tear or the wearing process, it is possible for the device to become pilled or discoloured particles, and the device being torn or damaged.

The guaranteed period of the device is 2 years without tearing or original properties, provided that the device is used in accordance with the instructions and product properties.

Resale packaging can be dangerous for children. After removing the device, the packaging should be removed so as not to let the child of the device.

In the event of allergic reactions or any other adverse reactions associated with use of the device, discontinue use immediately and consult a doctor.

STORAGE AND MAINTENANCE

The product is intended for use in the original packaging. During transport and storage, the product should be protected against dirt, dust, moisture and mechanical damage. Store the product at temperatures between 17°C and 25°C. The product should be stored in a dry, well-ventilated place. The product should be protected from the effects of temperatures and moisture. Protect the device from excessive exposure to dust, direct sunlight and heat sources (e.g., radiators, heating boilers). The device should be protected at least 2 maintenance periods are stored in the device in a sealed or closed container. Do not touch or use other parts of the device after the washing process, the duvet must be firmly sealed. The following maintenance periods parameters should be used:

NOTE: In order to eliminate allergens, the device should be washed periodically at high temperatures and periodically subjected to high temperature pressing (the use of steamers is also recommended).

COMPOSITION

Coat 100% polyester

Wash 100% polyester

EXPLANATION OF SYMBOLS USED ON DEVICE LABELS

CE CE marking conforming to the European Union (EU) Directive 93/68/EEC of the European Parliament and of the Council.

Consult instructions for use

Keep away from sunlight

REF Catalogue number

LOT Lot number

Store in dry place

Temperature limit

EXPLANATION OF SYMBOLS INDICATING THE DEVICE MAINTENANCE PROCESS

NO Maximum washing temperature 60°C washing process

Tumble drying prohibited, drying at low temperature, including ironing

Do not bleach

Do not iron

NO Do not bleach

NO Do not dry clean

NO Do not iron

In case of any serious incident related to the device, the manufacturer should be immediately informed by writing to the following address: "John Cooper Europe" S.p. z o.o., ul. John Coopera 20-26, 55-210 Szczecinek, Poland, by phone: +48 66 66 66 66 or electronically at info@johncooper.eu. It is also necessary to notify the competent authority of the Member State on whose territory the incident occurred.

"John Cooper Europe" S.p. z o.o.
ul. John Coopera 20-26, 55-210 Szczecinek, Poland
+48 66 66 66 66
info@johncooper.eu

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